Dehidrogenirani i-alkoholi, koji su dehidrogenirani alkoholi.

DE FD 333 Schnelldesinfektion

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EN FD 333 Quick-acting disinfection

FD 333 is an aldehyde-free ready-for-use solution for quick-acting spray and wipe disinfection and cleaning of surfaces of medical

devices (dental chairs, mobile trolleys, instrument cabinets, hand-pieces, angular pieces, protective covers of intraoral films after exposure, operating lamp handles, etc.).

Composition: 100 g FD 333 contain 62 g ethanol, 0.05 g didecyl-dimethyl-ammoniumchloride, auxiliaries and water.

Effective spectrum: Bactericidal, tuberculocidal, fungicidal¹⁾, virucidal activity²⁾ (encapsulated viruses such as vaccinia viruses, incl. HBV, HCV, HIV, SARS corona viruses³⁾ as well as non-encapsulated viruses, such as adeno viruses, polyoma viruses SV 40, polio viruses, noro viruses⁴⁾).
List of VAH/DGHM. Tested in accordance with EN 13727, EN 13624, EN 14348, EN 14476.

1) Testing with Candida albicans, Aspergillus niger.

2) In accordance with the recommendations of the RKI (Robert Koch Institute) (Bundesgesundheitsbl. 47, 62-66, 2004).

3) Test for BCV surrogate virus.

4) Test for MNV surrogate virus.

Directions for use: Pour FD 333 into the spray bottle. For disinfecting surfaces and articles, spray until fully wet and let dry or spray into cloth and wipe. The exposure time is 1 minute. For the inactivation of viruses we recommend 5 minutes (see Table in „DE“).

Caution: The material compatibility was tested and confirmed by well-known manufacturers. Take care with alcohol-sensitive material (e.g. acrylic glass). Flash point is 22 °C.

 see imprint.	
Danger. Highly flammable liquid and vapour. Causes serious eye irritation. Keep away from heat/sparks/open flames/hot surfaces. - No smoking. Wear protective gloves/eye protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store in a well-ventilated place. Keep cool. Dispose of contents/container to hazardous or special waste collection point.	
DÜRR DENTAL (Products) UK Ltd. 14 Linnwell Way – Telford Way Industrial Estate Kettering, Northants NN16 8PS, United Kingdom ☎ +44 1536526740	

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